RT\_FLC\_Grande\_100107

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Direttiva	Anno rilascio	Contenuto
MDD Directive 93/42/EEC	1993	Council Directive for medical devices
Title 21 CFR 820 QSR	1996	Quality System Regulation
ISO 9001	2000	Quality Management Systems
ISO 13485	2003	System requirements for regulatory purposes
EN ISO 14155-1	2003	Clinical Investigation of medical devices for human subjects- Part1: general requirements
MedDev 2.7.1	2003	Guidance: Evaluation of clinical data
EN 550/ISO 11135	1994	Sterilization of medical devices- Validation and routine control for EO sterilization + tech. Corrigendum 1994
EN 556-1	2001	Sterilisation of medical devices - Requirements for medical devices to be designated 'Sterile' - Part 1: Requirements for terminally sterilised medical devices
EN 980	2003	Graphic symbols for medical devices
ISO 14971 ISO 14971/A1	2000 2003	Medical Devices- Application of risk management to medical devices
EN 868 -1	1997	Requirements for sterile products packaging
EN ISO 10555-1/A2	2004	Sterile single-use intravascular catheters- Part 1 general Requirements
EN ISO 10555-4 EN ISO 10555-4/AC	1997 2002	Sterile single-use intravascular catheters- Part 4 Balloon dilatation catheters
EN 1707	1996	Conical fittings with 6% (Luer) taper for syringes, needles and certain other medical equipment
EN ISO 10993-1	2003	Biological evaluation of medical devices: Part1 Evaluation and testing
EN ISO 10993-4	2002	Biological evaluation of medical devices: Part 4 Selection of tests for interactions with blood
EN ISO 10993-5	1999	Biological evaluation of medical devices: Part 5 Tests for in vitro cytotoxicity
EN ISO 10993-7	1995	Biological evaluation of medical devices: Part 7 Ethylene oxide sterilization residuals
EN ISO 10993-10	2002	Biological evaluation of medical devices: Part 10 Tests for irritation and delayed-type hypersensitivity
EN ISO 10993-12	2004	Biological evaluation of medical devices - Part 12: Sample preparation and reference materials
EN 1041	1998	Information supplied by the manufacturer with medical devices
ISO 11737-1	1995	Sterilization of medical devices -- Microbiological methods -- Part 1: Estimation of population of microorganisms on products
ISO11607	2003	Packaging for terminally sterilized medical devices

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# CERTIFICATE

## for the Quality Assurance System

As a notified body of the European Union (Reg. No. 0124) DEKRA Certification GmbH hereby approves the Quality Assurance System applied for design, manufacture and final inspection by the company

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Via Industria 26/28 • 25030 Torbole Casaglia (BS), Italy

Approval is based on the decision dated 12.01.2009 and the result of the report no. 50251-Z3-00 and is performed in accordance with the stipulations of

### Annex II, Section 3 of the Directive 93/42/EEC

of the Council dated June 14, 1993 governing medical devices. The certification is applicable to the devices specified in the Annex. The devices in question are subjected to testing and examination in accordance with Annex II, Section 3 of the Directive 93/42/EEC. The listed devices may be affixed with the CE marking indicated below.

Date of the first certification: 16.02.1998

Date of the last recertification: 16.02.2008

This certificate is valid until: 15.02.2013

Certificate-registration No.: 50251-16-05  
English version

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Date: 27.08.2009

Page 1 of 2

## Devices/device categories included in the certificate

### Class II a

**PTA catheter:** Amphirion Deep  
Sailor Plus  
Admiral Xtreme

**Peripheral guide wire:** Skipper Pass  
Skipper Deep

**High Pressure PTA Balloon Catheter:** REEF HP

### Class II b

**Peripheral stent system:** Invastent Onda  
Chromis Deep  
Scuba

**Peripheral self-expanding stent system:** Maris  
Maris Deep

**Renal stent system:** Hippocampus

**RF ablation catheters and needles:** Miras LC  
Miras LN  
Miras RC  
Miras TX  
Miras BN

### Class III

**PTA catheter:** Submarine Plus  
Submarine Rapido  
Pacific Xtreme

**PTCA catheter:** Avion Plus  
Avion Plus OTW  
Avion Bifurcation RX2  
Falcon CTO  
Falcon Forte  
Falcon Bravo  
Falcon Grande  
Falcon Piccolo

This document may only be reproduced and distributed completely!

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Date: 27.08.2009

Page 2 of 2

## Devices/device categories included in the certificate

### Class III

<b>Coronary guide wire:</b>	Skipper Skipper Race
<b>Clot extraction catheter:</b>	DIVER C.E. DIVER C.E. kit DIVER C.E. MAX
<b>Multipurpose catheter:</b>	DIVER
<b>Cerebral protection device:</b>	MO.MA MO.MA ULTRA
<b>Coronary stent system:</b>	Invastent Volo Skylor Co-Cr stent
<b>Coronary bifurcation stent system:</b>	Twin Rail
<b>Carotid self-expanding stent system:</b>	Cristallo Ideale

ServiTech s.r.l.  
L'Amministratore

This document may only be reproduced and distributed completely!

# EC design examination certificate in accordance with Annex II, Section 4 of Directive 93/42/EEC

As a notified body of the European Union (Reg. No. 0124), DEKRA Certification GmbH hereby certifies for the manufacturer

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that the design dossier for the product(s) described in the Annex is in compliance with the pertinent stipulations of Council Directive 93/42/EEC dated June 14, 1993 governing medical devices.

The certificate is based on the report specified in the annex.

Product:

**PTCA Catheter:**

**Falcon CTO, Falcon Forte,**

**Falcon Bravo, Falcon Piccolo, Falcon Grande**

Date of the first certification: 08.02.2007

This certificate is valid until: 15.02.2013

Date of the last recertification: 16.02.2008

Certificate-registration No.: 50251-23-M5  
English version

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ServiTech s.r.l.  
L'Amministrazione

# Annex to the EC design examination certificate 50251-23-M5 dated 21.04.2009 English version

Revision status: 1

Date: 21.01.2010

Page 1 of 6

**Report number:** 50251-P14-07

**Product:** PTCA Catheter:  
Falcon CTO, Falcon Forte, Falcon Bravo, Falcon Piccolo, Falcon Grande

### Intended use:

Description of intended use: The Falcon balloon catheters are intended to be used in the coronary native arteries and bypass grafts like SVG (saphenous vein graft) and IMA (internal mammary artery) as well as in peripheral vascular districts like renal arteries and supra-aortic vessels (i.e. carotid arteries). They are indicated for the dilatation of stenotic portions and also for post dilatation of deployed stents.

### Technical specifications:

	Falcon Forte	Falcon CTO	Falcon Bravo
Guide wire design	<b>RX</b>	<b>RX and OTW</b>	
Balloon material	FLEXITEC™ HP with LFC hydrophilic coating	FLEXITEC™ PF with LFC hydrophilic coating	FLEXITEC™ HF with LFC hydrophilic coating
Balloon diameter (mm)	2,0 / 2,25 / 2,5 / 2,75 / 3,0 / 3,25 / 3,5 / 3,75 / 4,0 / 4,5 / 5,0	1,0 / 1,25 / 1,5	1,5 / 2,0 / 2,25 / 2,5 / 2,75 / 3,0 / 3,25 / 3,5 / 3,75 / 4,0
Balloon Length (mm)	7, 10, 14, 20, 30	10, 14, 20	0, 10, 14, 20, 30, 40
Catheter length, usable (cm)	145	145, 150 and 160	145 and 160
Shaft diameter (F)	Proximal: 2,0 (0,67mm) Distal: 2,7	<b>RX:</b> Proximal: 2,0 (0,67 mm) Distal: 2,7 taped down to 2,2 <b>OTW:</b> Proximal: 3,3 (1,12 mm) Distal: 2,7 taped down to 2,2	<b>RX:</b> Proximal: 2,0 (0,67 mm) Distal: 2,5 <b>OTW:</b> Proximal: 3,3 (1,12 mm) Distal: 2,5
Sterilization method	ETO-Sterilization		

ServiTech S.r.l.  
L'Amministratore

This document may only be reproduced and distributed completely!

**Annex to the EC design examination certificate 50251-23-M5**  
**dated 21.04.2009** English version

Revision status: 1

Date: 21.01.2010

Page 2 of 6

**Report number:** 50251-P14-07

**Product:** PTCA Catheter:  
 Falcon CTO, Falcon Forte, Falcon Bravo, Falcon Piccolo, Falcon Grande

**Technical specifications:**

	Falcon Grande	Falcon Piccolo
Guide wire design	RX	RX and OTW
Balloon material	FLEXITEC™ HS with LFC hydrophilic coating	FLEXITEC™ PF with LFC hydrophilic coating
Balloon diameter (mm)	4,5 / 5,0 / 5,5 / 6,0 / 7,0	1,25
Balloon Length (mm)	14, 20, 30, 40	10, 14, 20
Catheter length, usable (cm)	140	145, 150 and 160
Shaft diameter (F)	Proximal: 2,3 (0,76mm) Distal: 3,5	<b>RX:</b> Proximal: 2,0 (0,67 mm) Distal: 2,7 tapered down to 2,2 <b>OTW:</b> Proximal: 3,3 (1,12 mm) Distal: 2,7 tapered down to 2,2
Sterilization method	ETO-Sterilization	

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# CERTIFICATE

No. Q1N 08 11 57794 002

**Holder of Certificate:** Invatec Technology Center GmbH

Hungerbuelstrasse 12A  
8500 Frauenfeld  
SWITZERLAND

**Facility(ies):**

Invatec Technology Center GmbH  
Hungerbuelstrasse 12A, 8500 Frauenfeld, SWITZERLAND

**Certification Mark:**

**Scope of Certificate:**

Design and Development, Production of Vascular and Non - Vascular Stents and Catheters

**Applied Standard(s):**

EN ISO 13485:2003/ AC:2007  
Medical Devices - Quality Management Systems - Requirements for regulatory purposes  
Medizinprodukte - Qualitätsmanagementsysteme - Anforderungen für regulatorische Zwecke

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality system which meets the requirements of the listed standard(s). See also notes overleaf.

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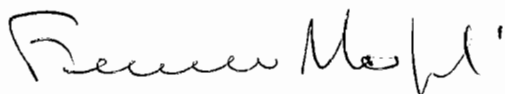
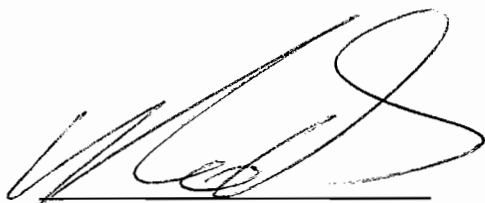
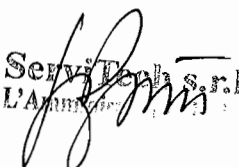
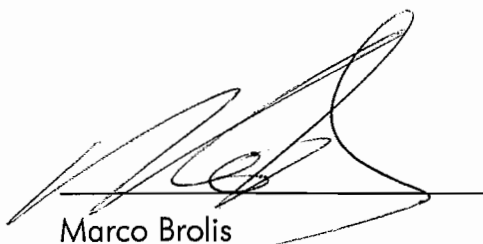
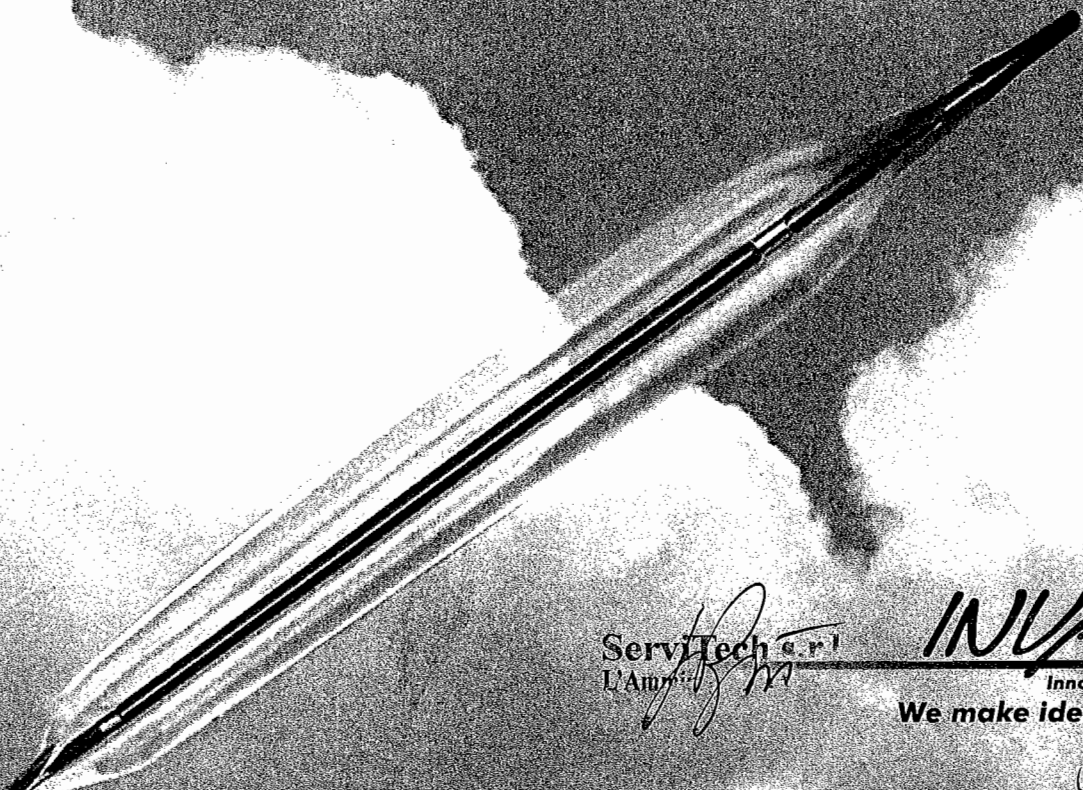
ServiTech srl  
L'Amministratore

**INVAtec**  
Innovative Technologies  
We make ideas come alive

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