Spett.le  
SO.RE.SAS. S.P.A.  
**C.a. Uff. Protocollo**  
Isola C1, Torre Saverio  
80143 NAPOLI  
Italy

10/9/2010

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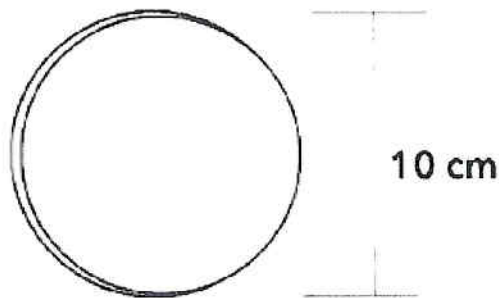
GPN REF G52406 LOT NXXXXX

CGML0907

**Cook Vascular Inc.**  
1186 Montgomery Lane  
Vandergrift, PA 15690  
Made in USA


EC REP **William Cook Europe ApS**  
Sandet 6, DK-4632 Bjæverskov  
DENMARK

FM-2857C

# Lead Extraction<sup>™</sup> Products

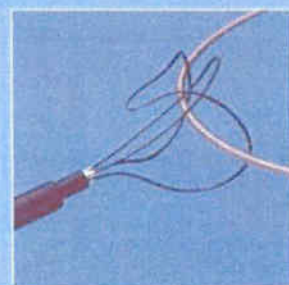and Accessories

**COOK<sup>®</sup>**  
MEDICAL


## LIBERATOR<sup>®</sup> Universal Locking Stylet

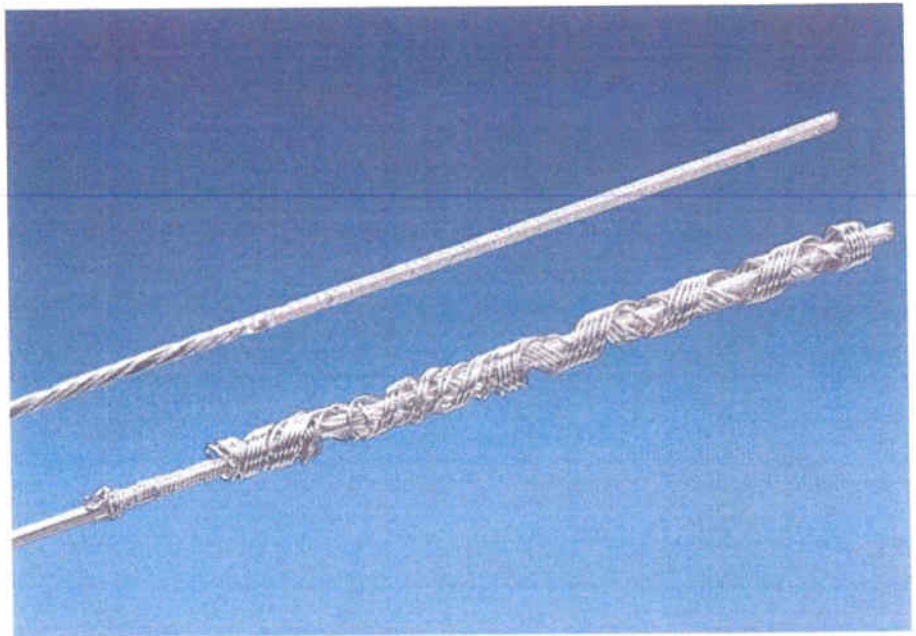Order Number: [G26550 \(LR-OFA01\)](#)

Intended for use in patients requiring the percutaneous removal of cardiac leads, indwelling catheters and foreign objects with a central lumen.

A specialized stylet wire that can be inserted through a cardiac lead's inner conductor lumen once the proximal connector has been removed. The stylet can then be locked into position within the lead's distal portion allowing for the delivery of tractional forces helpful during the extraction procedure. One size fits all leads with inner coil lumens. The Liberator Locking Stylet fits within and firmly grasps leads with lumen diameters of .016 to .032 inches.

- Overall Length: 140 cm, active length: 70 cm
- Strong, positive grip
- Threadable handle

## BULLDOG<sup>™</sup> Lead Extender

Order Number: [G46542 \(LR-LED01\)](#)

Designed to aid in the removal of lumenless cardiac leads or those that contain cable conductor wires by extending the targeted device's length during the extraction process so that dilator sheath sets can be used.

- Secure and easy to use locking mechanism
- Compatible with all Cook dilator sheath sets
- Threadable handle

Supplied sterile in peel-open package. Intended for one-time use.

## EVOLUTION™ Mechanical Dilator Sheath Set

- Threaded barrel distal tip design
- Rotating sheath aggressively negotiates fibrous binding sites often problematic for powered devices
- Stainless steel sheath braid for kink resistance and enhanced torque delivery
- Intuitive design is minimally dependent on physician technique
- Multiple I.D. sizes: 7, 9, 11, and 13 Fr
- No electrical energy supply needed

Order Number	Inner Diameter Size (Fr)
G46538 (LR-EVN-7.0)	7.0
G46539 (LR-EVN-9.0)	9.0
G46540 (LR-EVN-11.0)	11.0
G46541 (LR-EVN-13.0)	13.0

Note: Outer Sheath packaged with set and also available separately

These products are for use in the percutaneous dilation of tissue surrounding cardiac leads, indwelling catheters and foreign objects.

## Byrd Dilator Sheath Sets. Radiopaque. Intended for one time use. Individually packaged, sterile.

	Order Number	Length	Color	Inner Sheath		Outer Sheath		Size
				I.D.	O.D.	I.D.	O.D.	
Polypropylene	G26562 (LR-PPLBES-7.0)	Regular	Blue	7.0 Fr	9.3 Fr	10.0 Fr	12.3 Fr	AA
	G26563 (LR-PPLBES-7.0-XL)	Extra Long	Blue	7.0 Fr	9.3 Fr	10.0 Fr	12.3 Fr	
	G19893 (LR-PPLBES-8.5)	Regular	Yellow	8.5 Fr	10.7 Fr	11.6 Fr	13.9 Fr	A
	G20534 (LR-PPLBES-8.5-XL)	Extra Long	Yellow	8.5 Fr	10.7 Fr	11.6 Fr	13.9 Fr	
	G19859 (LR-PPLBES-10.0)	Regular	Green	10.0 Fr	12.1 Fr	13.1 Fr	15.2 Fr	B
	G20535 (LR-PPLBES-10.0-XL)	Extra Long	Green	10.0 Fr	12.1 Fr	13.1 Fr	15.2 Fr	
	G19860 (LR-PPLBES-11.5)	Regular	White	11.5 Fr	13.6 Fr	14.1 Fr	16.3 Fr	C
	G20525 (LR-PPLBES-11.5-XL)	Extra Long	White	11.5 Fr	13.6 Fr	14.1 Fr	16.3 Fr	
	G26514 (LR-PPLBES-13.0)	Regular	Orange	13.0 Fr	15.3 Fr	16.0 Fr	18.1 Fr	D
	G20526 (LR-PPLBES-13.0-XL)	Extra Long	Orange	13.0 Fr	15.3 Fr	16.0 Fr	18.1 Fr	

Teflon	G19999 (LR-TEFBES002)	Regular	Black	8.5	12.0	12.8 Fr	16.0 Fr	B
	G26496 (LR-TEFBES002-XL)	Extra-long	Black	8.5	12.0	12.8 Fr	16.0 Fr	
	G20221 (LR-TEFBES003)	Regular	Black	9.5	13.0	14.3 Fr	18.0 Fr	C
	G26497 (LR-TEFBES003-XL)	Extra-long	Black	9.5	13.0	14.3 Fr	18.0 Fr	

Regular length 13"/15" (33cm/38cm) Extra long length 16"/18" (41cm/46cm)

	Order Number	Color	Inner Sheath		Outer Sheath		Size
			I.D.	O.D.	I.D.	O.D.	
Telescoping Stainless Steel	G20536 (LR-TELSST001)	Metallic	8.7	10.3	11.5	12.6	A
	G20000 (LR-TELSST002)	Metallic	10.4	11.9	12.7	13.7	B
	G26506 (LR-TRPSST002)	Metallic	8.1	Functions as innermost sheath (3rd) for size B			
	G20537 (LR-TELSST003)	Metallic	12.3	13.8	14.5	15.6	C

A stainless steel outer sheath collar provides better control during sheath manipulation.

# PERFECTA<sup>®</sup> Electrosurgical Dissection System\*

The PERFECTA<sup>®</sup> EDS provides lead removal physicians with the powered means to separate, with directed precision, a transvenous lead from its fibrous binding attachments. Its dual electrode system performs as a bipolar dissection sheath and also functions as a mechanical dissection sheath. **RF** energy conducted between bipolar tungsten electrodes is used to locally dissect the binding tissue that surrounds and anchors transvenous leads. The tapered distal end of the EDS can also be used to mechanically disrupt these adhesions.

Now with the option of pulsed activation, the PERFECTA EDS allows the employment of an intermittent discrete **RF** dissecting arc between the bipolar electrodes located at the sheath's distal end. The inclusion of our PERFECTA Ergonomic Intermittent Activation Switch gives you a choice of continuous or intermittent power.

*Provides the lead removal physician with the means to separate, with directed precision, a transvenous lead from its fibrous binding attachments.*

\*Intended to be used exclusively with either the Valley Lab Force FX-C or Force FX-8 electro-surgical generators.

Complete  
PERFECTA<sup>®</sup>  
Electrosurgical  
Dissection  
System

PERFECTA<sup>®</sup> EDS  
Order Number

G26581 (LR-PFTA-7.0)  
G26584 (LR-PFTA-9.0)  
G26579 (LR-PFTA-11.0)  
G26583 (LR-PFTA-13.0)

Cutting action limited to that area between the electrodes. "Localized dissection" can help to avoid vessel trauma and thrombosis seen with aggressive circumferential ablation.

The marriage of a proven mechanical extraction tool with a technologically-advanced energized device.

**PERFECTA® EDS Ergonomic Intermittent Activation Switch**

Order Number: [G31600 \(LR-PFTAIC-02\)](#)

**PERFECTA® Interface Adapter**

Order Number: [G26582 \(LR-PFTAIFB-02\)](#)

The uniquely designed Electrosurgical Dissection System Interface Adapter allows the EDS to function properly when connected to an appropriate electrosurgical generator. Compatible Generators: Valleylab Force FX™-C, Valleylab Force FX™-8.

Order Number: [G31625 \(LR-PFTA-STARTER-02\)](#)

**System Contents:**

- Interface Adapter
- Ergonomic Intermittent Activation Switch
- Storage Pouch
- 2 of each sheath sets
- 2 of each outer sheath model

**Electrosurgical Dissection Sheath w/outer sheath**

Conductor Color	Inner Sheath		Outer Sheath	
	I.D.	O.D.	I.D.	O.D.
Green	7 Fr	11 Fr	12 Fr	15 Fr
Black	9 Fr	13 Fr	14 Fr	17 Fr
Blue	11 Fr	15 Fr	16 Fr	19 Fr
Orange	13 Fr	17 Fr	18 Fr	21 Fr

**Outer Sheath Only**

Order Number	Compatible with	Outer Sheath O.D.
<a href="#">G26580 (LR-PFTA-7.0-CS)</a>	LR-PFTA-7.0	15 Fr
<a href="#">G26578 (LR-PFTA-9.0-CS)</a>	LR-PFTA-9.0	17 Fr
<a href="#">G26586 (LR-PFTA-11.0-CS)</a>	LR-PFTA-11.0	19 Fr
<a href="#">G26585 (LR-PFTA-13.0-CS)</a>	LR-PFTA-13.0	21 Fr

**Superior Approach Accessories**

**LEAD CLIPPERS**

Order Number: [G20003 \(LR-CLP001\)](#)

Used to separate connector from pacemaker or defibrillator lead wire.

- Cleanly cuts the lead so that the inner coil is exposed.
- Individually packaged, sterile

**COIL EXPANDER**

Order Number: [G20005 \(LR-CEX001\)](#)

For use in preparing the cut lead coil to allow for proper sizing using gauge pins.

- Individually packaged, sterile

**STYLET WIRES**

Order Number: [G20006 \(LR-STS001\)](#)

Use to debride foreign matter inside lead body, and to measure length of lead after connector has been removed.

- Insert stylet wire into lead to confirm patency
- Provided as a pair: 60 cm (for original locking stylets) and 65 cm (for LIBERATOR™ and WILKOFF™ Locking Stylets).
- .015 inch diameter.
- Individually packaged, sterile

**SOF-GRIP HEMOSTAT**

Order Number: [G20001 \(LR-SGH001\)](#)

For use in gripping the pacing or defibrillator lead without damaging the lead.

- Individually packaged, sterile

**PIN VISE**

Order Number: [G20004 \(LR-PV1001\)](#)

Used for extra strength and comfort in the gripping of selected sheath sets.

- Designed to aid in the delivery of torque to plastic Byrd Dilator Sheath Sets.
- Slide Pin Vise over sheath and tighten. Sheath will not be damaged.
- Individually packaged, sterile
- Hollow to allow passage of locking stylet

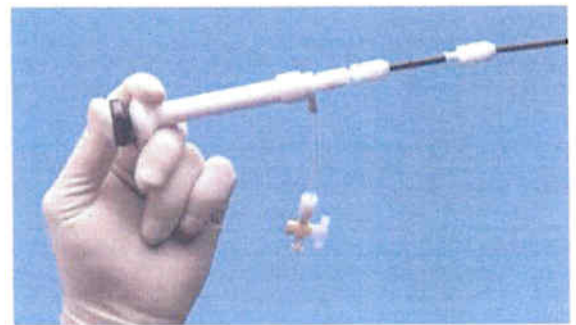
Thoroughly Read and Understand the Suggested Instructions for Use insert that accompanies the various LEAD EXTRACTION products prior to using these devices. Rx Only.

## Needles Eye<sup>®</sup> Snare

For use in the percutaneous retrieval of indwelling catheters, cardiac leads, fragments of catheter tubing or wire guides, and other foreign objects.

A transfemoral grasping device that forms a basket-like snare around the lead body. It is delivered to the vicinity of the lead through a long, flexible 12 Fr sheath placed coaxially within a larger outer 16 Fr (O.D.) sheath having a hemostasis valve at its proximal end. (Two sizes of distal grasping mechanism. Requires no extra handle.)

- Usable (shaft) Length: 94 cm (54 cm shortened version also available)
- Inner Sheath: I.D. 8.5 Fr—O.D. 12.0 Fr
- Outer Sheath: I.D. 12.8 Fr—O.D. 16.0 Fr
- Designed to grasp straight or configured objects with or without free ends
- Fabricated with a shape-retentive alloy for preservation of shape
- Provides quick-release capability by simply retracting the threader mechanism
- Is compatible with the Curved Inner Femoral Sheath (LR-CRVFEM001) for added positionability within the venous system
- Flush port is continuous with the snare's innermost lumen
- Intended for one-time use
- Individually packaged, sterile

94 cm Version	Order Number	Description	Profile Diameter	Package Contents
	G26517 (LR-NES001)	Complete Needle's Eye <sup>®</sup> Snare retrieval set	13mm	Snare, 16 Fr. Check-Flo <sup>®</sup> Introducer Set, 12 Fr. Straight Inner Femoral Sheath
	G26515 (LR-NES002)	Complete Needle's Eye <sup>®</sup> Snare retrieval set	20mm	Snare, 16 Fr. Check-Flo Introducer Set, 12 Fr. Straight Inner Femoral Sheath
	G26518 (LR-SSN001)	Needle's Eye <sup>®</sup> Snare and Straight Inner Femoral Sheath	13mm	Snare and inner sheath only
	G26516 (LR-SSN002)	Needle's Eye <sup>®</sup> Snare and Straight Inner Femoral Sheath	20mm	Snare and inner sheath only
<b>54 cm Version</b> (Shortened)	G26533 (LR-NES541)	Complete Needle's Eye <sup>®</sup> Snare retrieval set	13mm	Snare, 16 Fr. Check-Flo Introducer Set, 12 Fr. Straight Inner Femoral Sheath, and Curved Inner Femoral Sheath
<b>Components Available Separately for 94 cm version</b>	G26505 (LR-CRVFEM001)	Curved 12 Fr Inner Femoral Sheath		
	G26566 (LR-CSS16)	Curved 16 Fr Introducer Sheath Set-Femoral (also contains complete Seldinger venous access equipment)		

## Byrd Workstation™ Femoral Intravascular Retrieval Set

16 Fr (O.D.) thick-walled sheath that contains a coaxial 12 Fr cannula pre-loaded with a Dotter Helical Loop Basket and a Tip Deflecting Wire Guide. The pre-loaded cannula passes through the outer cannula to carry these two instruments, via the femoral vein, into the right atrium. (Requires a Handle for Tip Deflecting Wire Guide (LR-HDG001) sold separately). Complete Seldinger venous access equipment included.

Order Number: G20007 (LR-WSF001)

### Included Components

- 16 Fr Workstation Check-Flo Introducer Sheath Set
- 12 Fr Straight Inner Femoral Sheath
- Pre-Loaded Helical Loop Basket and Tip Deflecting Wire Guide
- (Handle for Tip Deflecting Wire Guide not included)

## Tip Deflecting Wire Guide

Order Number: G20009 (LR-DWG001)

A long, thin steel cable that can deflect its tip around an object that is to be removed from the heart. (Requires Handle for Tip Deflecting Wire Guide)

## Handle for Tip Deflecting Wire Guide

Order Number: G20010 (LR-HDG001)

When connected to the proximal end of the Tip Deflecting Wire Guide, this handle can be used to manipulate the distal end of the Tip Deflecting Wire Guide and lock it into position.

## Pre-Loaded Helical Loop Basket and Tip Deflecting Wire Guide

Order Number: G20008 (LR-BDG001)

A flexible 12 Fr sheath containing the Dotter Helical Loop Basket and Tip Deflecting Wire Guide. 16 Fr Seldinger access equipment not included.

Thoroughly Read and Understand the Suggested Instructions for Use insert that accompanies the various LEAD EXTRACTION products prior to using these devices. Rx Only.

Distributed by:

COOK MEDICAL INCORPORATED (USA only)  
750 Daniels Way, P.O. Box 4195, Bloomington, IN 47402-4195 USA  
Phone: 812 339-2235, Toll Free: 800-457-4500, Toll Free Fax: 800-554-8335

COOK (CANADA) INC.,  
111 Sandiford Drive, Stauffville, Ontario, L4A 7X5 CANADA  
Phone: 905-640-7110, Toll Free: 800-668-0300

WILLIAM A. COOK AUSTRALIA PTY. LTD.  
Brisbane Technology Park, 12 Electronics Street, Eight Mile Plains  
Brisbane QLD 4113 AUSTRALIA, Phone: +61 7 38 41 11 88

WILLIAM COOK EUROPE ApS  
Sandet 6, DK-4632, Bjaeverskov, DENMARK, Phone: +45 56 86 86 86

© COOK VASCULAR INCORPORATED 2007

[www.cookmedical.com](http://www.cookmedical.com)

# Lead Extraction<sup>™</sup> Products

and Accessories

**WILLIAM A. COOK AUSTRALIA PTY. LTD.**  
Brisbane Technology Park, 12 Electronics Street, Eight Mile Plains  
Brisbane QLD 4113 AUSTRALIA, Phone: +61 7 38 41 11 88  
**WILLIAM COOK EUROPE ApS**  
Sandet 6, DK-4632, Bjernesekov, DENMARK, Phone: +45 56 86 86 86  
© COOK VASCULAR INCORPORATED 2005

Distributed by:  
**COOK MEDICAL INCORPORATED (USA only)**  
750 Daniels Way, P.O. Box 4195, Bloomington, IN 47402-4195 USA  
Phone: 812 339-2235, Toll Free: 800 457-4500, Toll Free Fax: 800 554-8335  
**COOK (CANADA) INC.**  
111 Sandford Drive, Southville, Ontario, L4A 7X5 CANADA  
Phone: 905 640-7110, Toll Free: 800 666-0300

[www.cookmedical.com](http://www.cookmedical.com)

**COOK<sup>®</sup>**  
Vascular Products

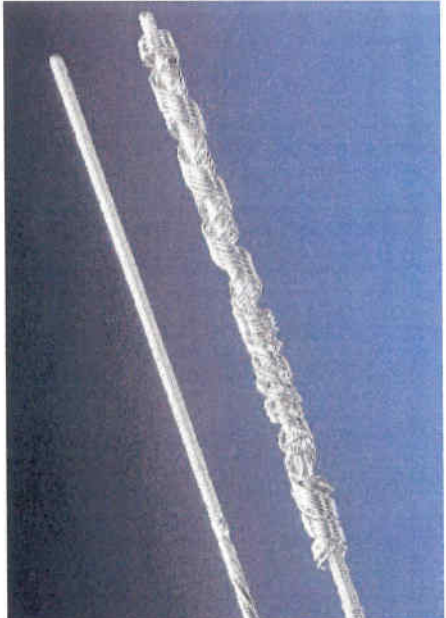
**COOK<sup>®</sup>**  
Vascular Products

For over 15 years, Cook has teamed with pioneering physicians to develop the most comprehensive line of both mechanical and powered tools and techniques for Lead Extraction™. Cook. No one offers more.

**LIBERATOR® Universal Locking Stylet**

Order Number: 626550 (LR-07A01)

Intended for use in patients requiring the percutaneous removal of cardiac leads, indwelling catheters and foreign objects with a central lumen. A specialized stylet wire that can be inserted through a cardiac lead's inner conductor lumen once the proximal connector has been removed. The stylet can then be locked into position within the lead's distal portion allowing for the delivery of tractional forces helpful during the extraction procedure. One size fits all leads with inner coil lumens. The Liberator Locking Stylet fits within and firmly grasps leads with lumen diameters of .016 to .032 inches.

- Overall Length: 140 cm, active length: 70 cm
- Strong, positive grip
- Threadable handle

**BYRD Dilator Sheath Sets**

**POLYPROPYLENE**

For use in the percutaneous dilation of tissue surrounding cardiac leads, indwelling catheters and foreign objects. A pair of long and moderately flexible cannulae, one fitting coaxially inside the other and capable of moving freely. The sheath set is advanced over the lead to break up fibrous adhesions. It also aids in "counter-traction" by stabilizing the heart wall when the lead tip is pulled out of the endocardial scar mass.

- Regular Length: 13"/15" (33cm/38cm)
- Extra Long Length: 16"/18" (41cm/46cm)
- Radiopaque
- Five color-coded French sizes of inner diameters
- Intended for one-time use
- Individually packaged, sterile

**TELESCOPING STAINLESS STEEL**

Designed to be replaced by appropriately sized Teflon Byrd Dilator Sheath Sets once venous access has been established. A stainless steel outer sheath collar provides better control during sheath manipulation.

- Intended for one-time use
- Individually packaged, sterile

Order Number	Length	Color	Inner Sheath		Outer Sheath		Size
			I.D.	O.D.	I.D.	O.D.	
625562 (LR-PPY-BES-7.0)	Regular	Blue	7.0 Fr	9.3 Fr	10.0 Fr	12.3 Fr	AA
625563 (LR-PPY-BES-7.0-XL)	Extra Long	Blue	7.0 Fr	9.3 Fr	10.0 Fr	12.3 Fr	AA
619893 (LR-PPY-BES-8.5)	Regular	Yellow	8.5 Fr	10.7 Fr	11.6 Fr	13.9 Fr	A
620034 (LR-PPY-BES-8.5-XL)	Extra Long	Yellow	8.5 Fr	10.7 Fr	11.6 Fr	13.9 Fr	A
619859 (LR-PPY-BES-10.0)	Regular	Green	10.0 Fr	12.1 Fr	13.1 Fr	15.2 Fr	B
620035 (LR-PPY-BES-10.0-XL)	Extra Long	Green	10.0 Fr	12.1 Fr	13.1 Fr	15.2 Fr	B
619860 (LR-PPY-BES-11.0)	Regular	White	11.5 Fr	13.6 Fr	14.1 Fr	16.1 Fr	C
620036 (LR-PPY-BES-11.0-XL)	Extra Long	White	11.5 Fr	13.6 Fr	14.1 Fr	16.1 Fr	C
626574 (LR-PPY-BES-13.0)	Regular	Orange	13.0 Fr	15.3 Fr	16.0 Fr	18.1 Fr	D
627026 (LR-PPY-BES-13.0-XL)	Extra Long	Orange	13.0 Fr	15.3 Fr	16.0 Fr	18.1 Fr	D

Order Number	Color	Inner Sheath		Outer Sheath		Size
		I.D.	O.D.	I.D.	O.D.	
620636 (LR-TELS1001)	Metallic	8.7	10.3	11.5	12.6	A
620600 (LR-TELS1002)	Metallic	10.4	11.9	12.7	13.7	B
620606 (LR-TWSS1000)	Metallic	8.1	10.1	11.5	12.6	B
620637 (LR-TELS1003)	Metallic	12.3	13.8	14.5	15.6	C

**TEFLON**

Teflon sheaths are more flexible than the Polypropylene Byrd Dilator Sheaths described above.

- Regular Length: 13"/15" (33cm/38cm)
- Extra Long Length: 16"/18" (41cm/46cm)
- Radiopaque
- Two French sizes (8.5 Fr and 9.5 Fr I.D.)
- Intended for one-time use
- Individually packaged, sterile

Order Number	Length	Color	Inner Sheath		Outer Sheath		Size
			I.D.	O.D.	I.D.	O.D.	
619899 (LR-TEFBS007)	Regular	Black	8.5	12.0	12.8	16.0	B
620496 (LR-TEFBS007-XL)	Extra-long	Black	8.5	12.0	12.8	16.0	B
620221 (LR-TEFBS009)	Regular	Black	9.5	13.0	14.3	18.0	C
620497 (LR-TEFBS009-XL)	Extra-long	Black	9.5	13.0	14.3	18.0	C

Thoroughly Read and Understand the Suggested Instructions for Use insert that accompanies the various LEAD EXTRACTION™ products prior to using these devices. Rx Only

## PERFECTA® Electrosurgical Dissection System\*

The PERFECTA® EDS provides lead removal physicians with the powered means to separate, with directed precision, a transvenous lead from its fibrous binding attachments. Its dual electrode system performs as a bipolar dissection sheath and also functions as a mechanical dissection sheath. RF energy conducted between bipolar tungsten electrodes is used to locally dissect the binding tissue that surrounds and anchors transvenous leads. The tapered distal end of the EDS can also be used to mechanically disrupt these adhesions.

Now with the option of pulsed activation, the PERFECTA EDS allows the employment of an intermittent discrete RF dissecting arc between the bipolar electrodes located at the sheath's distal end. The inclusion of our PERFECTA Ergonomic Intermittent Activation Switch gives you a choice of continuous or intermittent power.

Provides the lead removal physician with the means to separate, with directed precision, a transvenous lead from its fibrous binding attachments.

Cutting action limited to that area between the electrodes. "Localized dissection" can help to avoid vessel trauma and thrombosis seen with aggressive circumferential ablation.

The marriage of a proven mechanical extraction tool with a technologically-advanced energized device.

### PERFECTA® EDS Ergonomic Intermittent Activation Switch

Order Number: G31800 (LR-PFTAEG-02)

### PERFECTA® Interface Adapter

Order Number: G26562 (LR-PFTAFB-02)

The uniquely designed Electrosurgical Dissection System Interface Adapter allows the EDS to function properly when connected to an appropriate electrosurgical generator. Compatible Generators: Valleylab Force FX™-C, Valleylab Force FX™-8.

### Complete PERFECTA® Electrosurgical Dissection System

Order Number: G31825 (LR-PFTA-STARTER-02)

System Contents:

- Interface Adapter
- Ergonomic Intermittent Activation Switch
- Storage Pouch
- 2 of each sheath sets
- 2 of each outer sheath model

### PERFECTA® EDS

Order Number	Conductor Color	Inner Sheath I.D.	Outer Sheath O.D.
G26581 (LR-PFTA-7.0)	Green	7 Fr	11 Fr
G26584 (LR-PFTA-9.0)	Black	9 Fr	13 Fr
G26579 (LR-PFTA-11.0)	Blue	11 Fr	15 Fr
G26583 (LR-PFTA-13.0)	Orange	13 Fr	17 Fr

### Electrosurgical Dissection Sheath w/outer sheath

Order Number	Compatible with	Outer Sheath O.D.
G26580 (LR-PFTA-7.0-CS)	LR-PFTA-7.0	15 Fr
G26578 (LR-PFTA-9.0-CS)	LR-PFTA-9.0	17 Fr
G26586 (LR-PFTA-11.0-CS)	LR-PFTA-11.0	19 Fr
G26585 (LR-PFTA-13.0-CS)	LR-PFTA-13.0	21 Fr

## Superior Approach Accessories

### LEAD CLIPPERS

Order Number: G201003 (LR-CL-P001)

Used to separate connector from pacemaker or defibrillator lead wire.

- Clearly cuts the lead so that the inner coil is exposed.
- Individually packaged, sterile.

### COIL EXPANDER

Order Number: G200005 (LR-CE-M01)

For use in preparing the coil lead coil to allow for proper strapping using gauge pins.

- Individually packaged, sterile

### STYLET WIRES

Order Number: G201004 (LR-ST-M01)

Use to debrief foreign matter inside lead body, and to measure length of lead after connector has been removed.

- Insert stylet wire into lead to confirm patency.

- Provided as a pair: 60 cm (for original locking stylets) and 65 cm (for LIBERATOR® and WILKOFF® Locking Stylets).

- .015 inch diameter.

- Individually packaged, sterile

### SOFT-GRIP HEMOSTAT

Order Number: G200101 (LR-SRH001)

For use in grasping the pacing or defibrillator lead without damaging the lead.

- Individually packaged, sterile

### PIN VISE

Order Number: G200044 (LR-PV001)

Used for extra strength and comfort in the gripping of selected sheath sets.

- Designed to aid in the delivery of torque to plastic Byrd Dilator Sheath Sets.
- Slide Pin Vise over sheath and tighten. Sheath will not be damaged.
- Individually packaged, sterile
- Hollow - to allow passage of locking stylet

COOK®

Thoroughly Read and Understand the Suggested Instructions for Use Insert that accompanies the various LEAD EXTRACTION™ products prior to using these devices. Rx Only.

### Needle's-Eye® Snare

For use in the percutaneous retrieval of indwelling catheters, cardiac leads, fragments of catheter tubing or wire guides, and other foreign objects.

A transfemoral grasping device that forms a basket-like snare around the lead body. It is delivered to the vicinity of the lead through a long, flexible 12 Fr sheath placed coaxially within a larger outer 16 Fr (O.D.) sheath having a hemostasis valve at its proximal end. (Two sizes of distal grasping mechanism. Requires no extra handle.)

- Usable (shaft) Length: 94 cm (54 cm shortened version also available)
- Inner Sheath: I.D. 8.5 Fr—O.D. 12.0 Fr
- Outer Sheath: I.D. 12.8 Fr—O.D. 16.0 Fr
- Designed to grasp straight or configured objects with or without free ends fabricated with a shape-retentive alloy for preservation of shape
- Provides quick-release capability by simply retracting the threader mechanism
- Is compatible with the Curved Inner Femoral Sheath (LR-CRIFEM001) for added positionability within the venous system
- Flush port is continuous with the snare's innermost lumen
- Intended for one-time use.
- Individually packaged, sterile

94 cm Version	Order Number	Description	Profile Diameter	Package Contents
	G26517 (LR-HES001)	Complete Needle's-Eye® Snare retrieval set	13mm	Snare, 16 Fr. Check-Flo® Introducer Set, 12 Fr. Straight Inner Femoral Sheath
	G26518 (LR-HES002)	Complete Needle's-Eye® Snare retrieval set	20mm	Snare, 16 Fr. Check-Flo Introducer Set, 12 Fr. Straight Inner Femoral Sheath
	G26519 (LR-SSN001)	Needle's-Eye® Snare and Straight Inner Femoral Sheath	13mm	Snare and inner sheath only
	G26516 (LR-SSN002)	Needle's-Eye® Snare and Straight Inner Femoral Sheath	20mm	Snare and inner sheath only
54 cm Version (Shortened)	G26533 (LR-HES041)	Complete Needle's-Eye® Snare retrieval set	13mm	Snare, 16 Fr. Check-Flo Introducer Set, 12 Fr. Straight Inner Femoral Sheath, and Curved Inner Femoral Sheath
Components Available Separately for 94 cm version	G26506 (LR-CRIFEM001)	Curved 12 Fr Inner Femoral Sheath	13mm	91.4 cm
	G26566 (LR-CSS16)	Curved 16 Fr Introducer Sheath Set-Femoral (also contains complete Seldinger venous access equipment)	27mm	88.6 cm

### Byrd Workstation™ Femoral Intravascular Retrieval Set

16 Fr (O.D.) thick-walled sheath that contains a coaxial 12 Fr cannula pre-loaded with a Dotter Helical Loop Basket and a Tip Deflecting Wire Guide. The pre-loaded cannula passes through the outer cannula to carry these two instruments, via the femoral vein, into the right atrium. (Requires a Handle for Tip Deflecting Wire Guide (LR-HDC001) sold separately.) Complete Seldinger venous access equipment included.

- Order Number: G20007 (LR-WRF001)
- Included Components
- 16 Fr Workstation Check-Flo Introducer Sheath Set
  - 12 Fr Straight Inner Femoral Sheath
  - Pre-Loaded Helical Loop Basket and Tip Deflecting Wire Guide
  - (Handle for Tip Deflecting Wire Guide not included)

### Tip Deflecting Wire Guide

A long, thin steel cable that can deflect its tip around an object that is to be removed from the heart. (Requires Handle for Tip Deflecting Wire Guide)

### Handle for Tip Deflecting Wire Guide

When connected to the proximal end of the Tip Deflecting Wire Guide, this handle can be used to manipulate the distal end of the Tip Deflecting Wire Guide and lock it into position.

### Pre-Loaded Helical Loop Basket and Tip Deflecting Wire Guide

A flexible 12 Fr sheath containing the Dotter Helical Loop Basket and Tip Deflecting Wire Guide. 16 Fr Seldinger access equipment not included.

Thoroughly Read and Understand the Suggested Instructions for Use insert that accompanies the various LEAD EXTRACTION™ products prior to using these devices. Bx Only.

Spett.le  
SO.RE.SAS. S.P.A.  
**C.a. Uff. Protocollo**  
Isola C1, Torre Saverio  
80143 NAPOLI  
Italy

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